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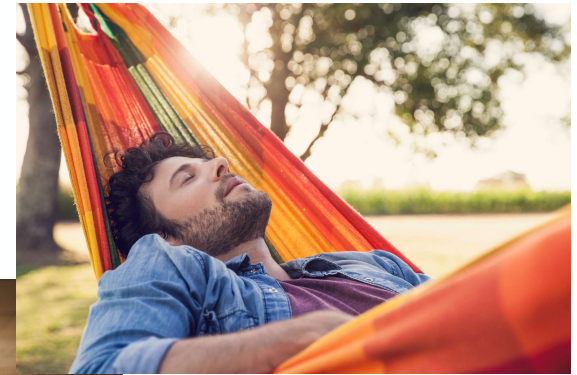
Therapy settings associated with optimal outcomes for t:slim X2 with Control-IQ technology in real world clinical care

Messer LH and Breton MD. Therapy settings associated with optimal outcomes for t:slim X2 with Control-IQ technology in real world clinical care. *Diabetes Technol Ther.* 2023;ahead of print. doi: 10.1089/dia.2023.0308.

positively different

Confidential » Not for Distribution
© 2022 Tandem Diabetes Care, Inc. All Rights Reserved

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- + Dexcom G6
- + Dexcom G7

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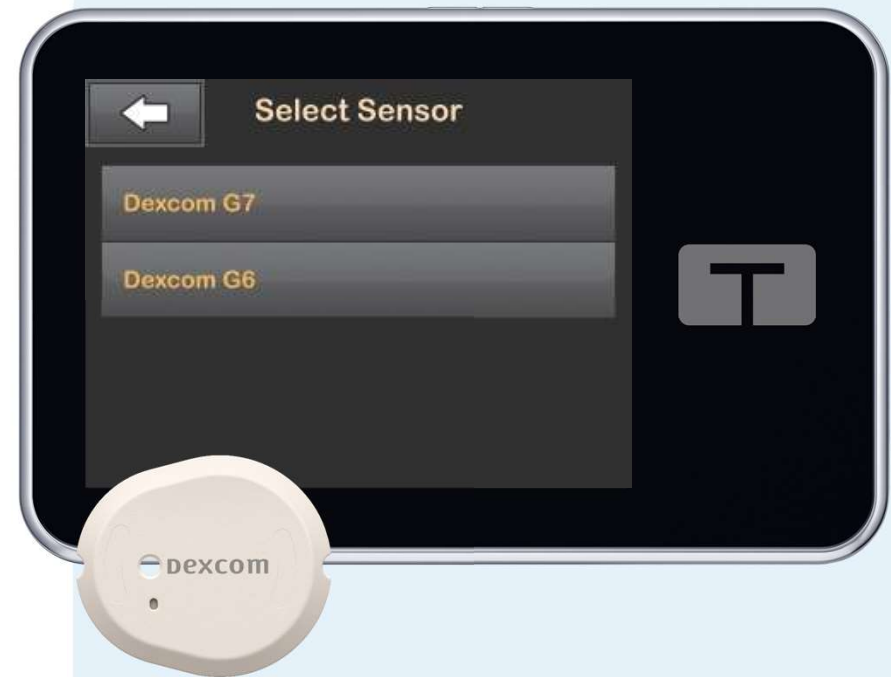
Confidential »» Not for Distribution
© 2023 Tandem Diabetes Care, Inc. All Rights Reserved

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

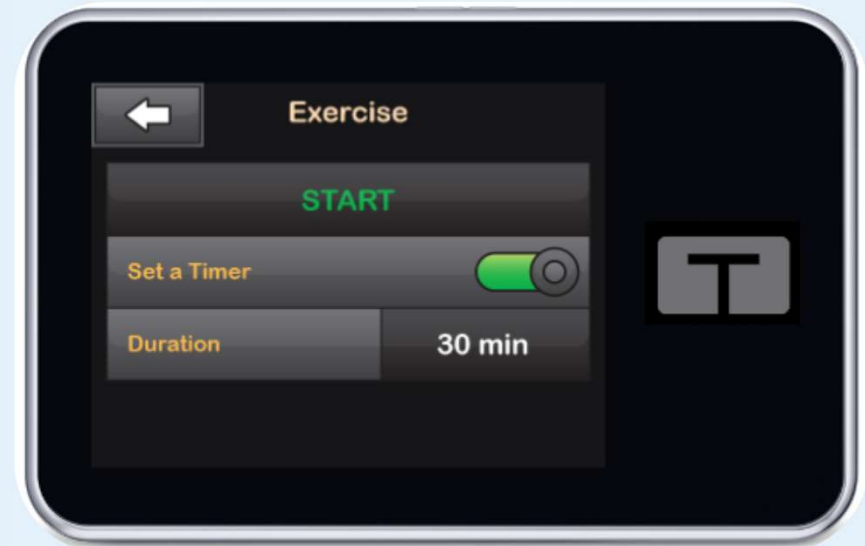
Tandem Source et t:slim mobile app

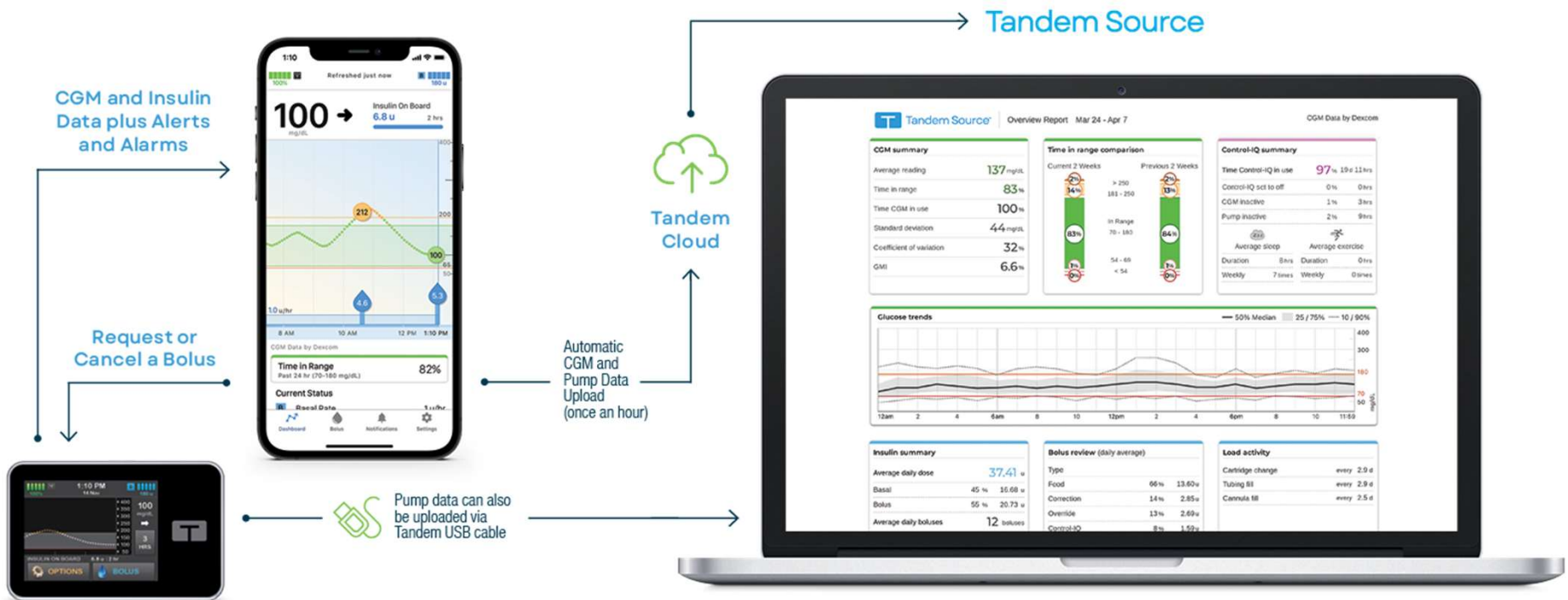Tandem t:slim mobile app

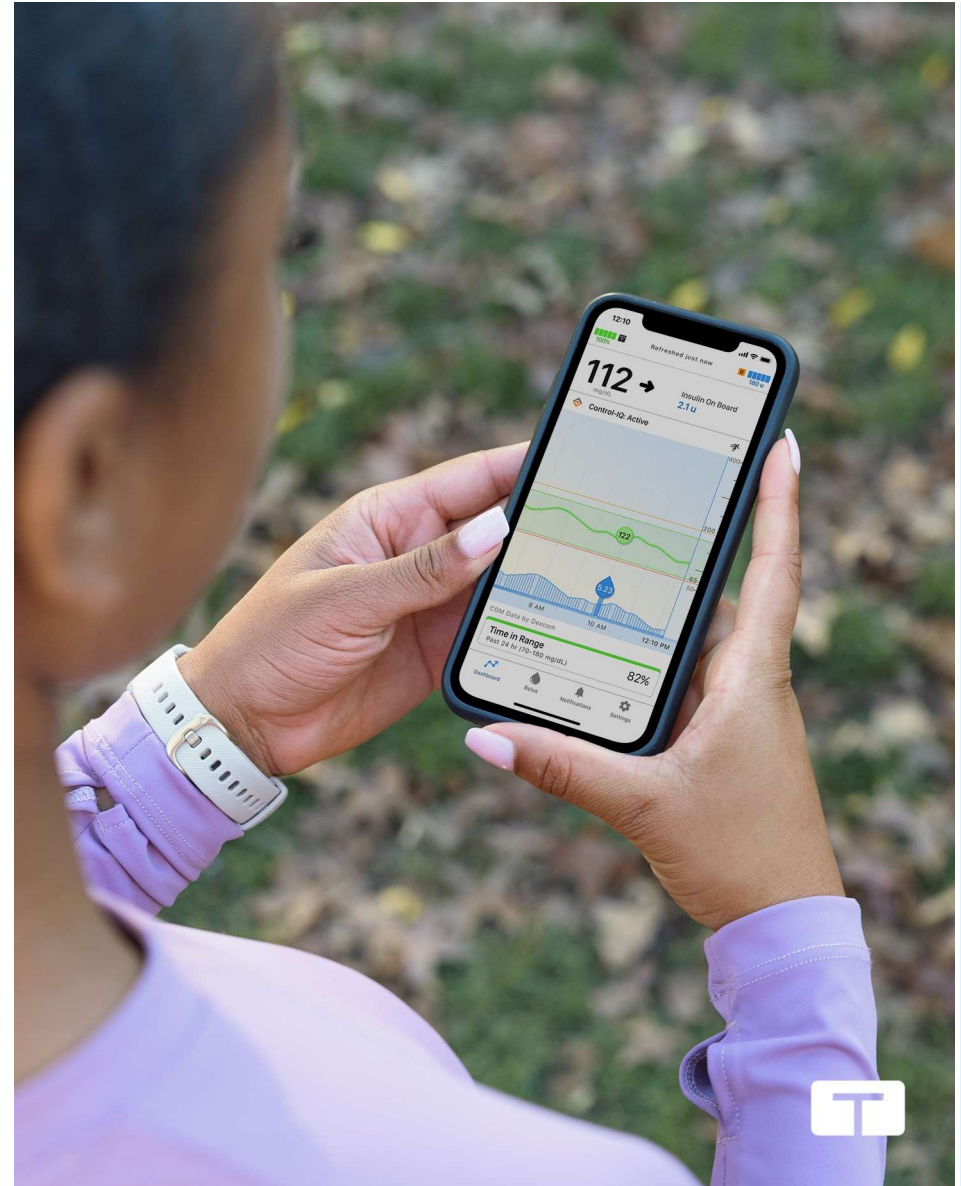Confidential »» Not for Distribution
© 2022 Tandem Diabetes Care, Inc. All Rights Reserved

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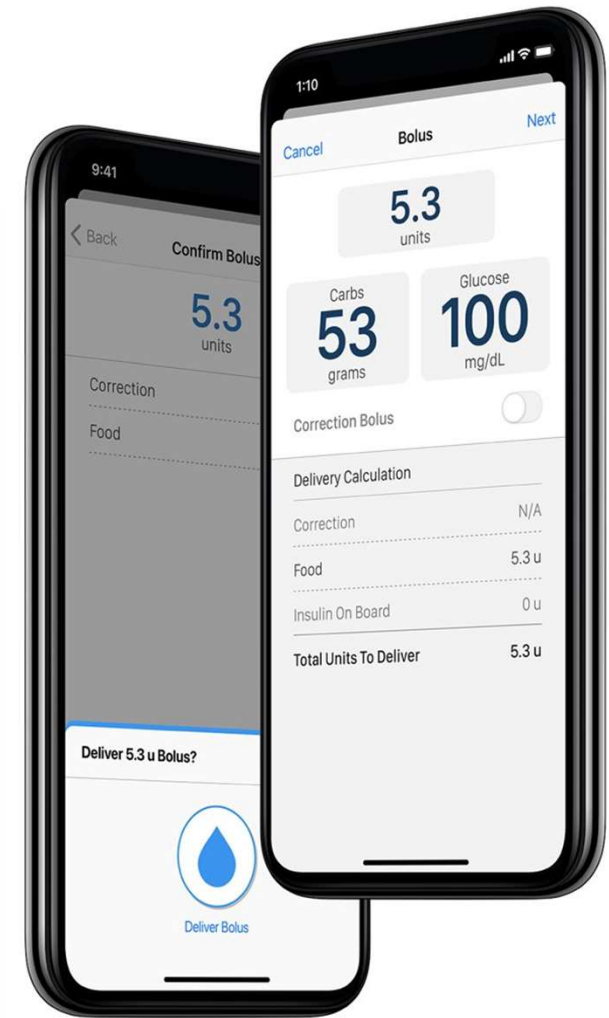
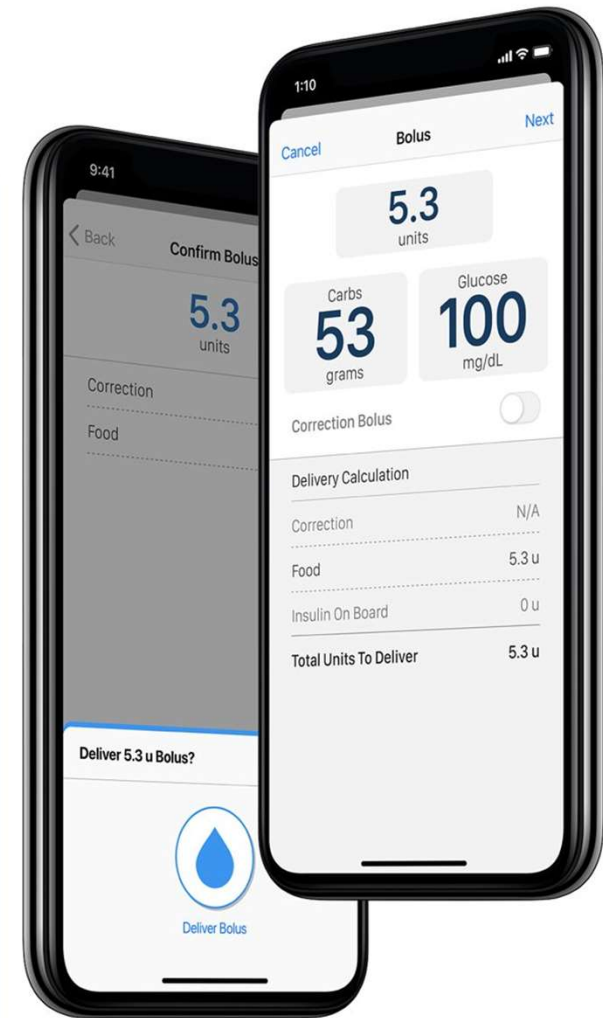
Age group	N	Number of user-initiated boluses (median [interquartile range])		Median number of boluses delivered by smartphone bolus (post upgrade)	Change in boluses	
		Pre smartphone bolus	Post smartphone bolus		Δ Number of bolus	% Δ Number of bolus
Overall	4,470	2.2 (1.6–2.6)	2.7 (2.0–3.3)	1.0 (0.4–1.7)	0.48***	22.1%
<12	43	2.5 (2.1–2.9)	3.0 (2.4–3.7)	0.7 (0.3–1.6)	0.45***	17.9%
12–25	1,283	2.1 (1.5–2.6)	2.6 (1.9–3.3)	1.0 (0.5–1.8)	0.47***	22.2%
26–64	2,852	2.2 (1.6–2.6)	2.7 (2.0–3.4)	0.9 (0.4–1.7)	0.52***	23.5%
65+	292	2.4 (1.8–2.7)	2.9 (2.3–3.4)	1.1 (0.3–2.0)	0.53***	22.6%

Table I. Number of User-Initiated Boluses Given Before and After Adoption of the Smartphone Bolus Feature of the t:connect Mobile App in People Who Bolused Less Than Times Per Day Prior to Mobile App Update.

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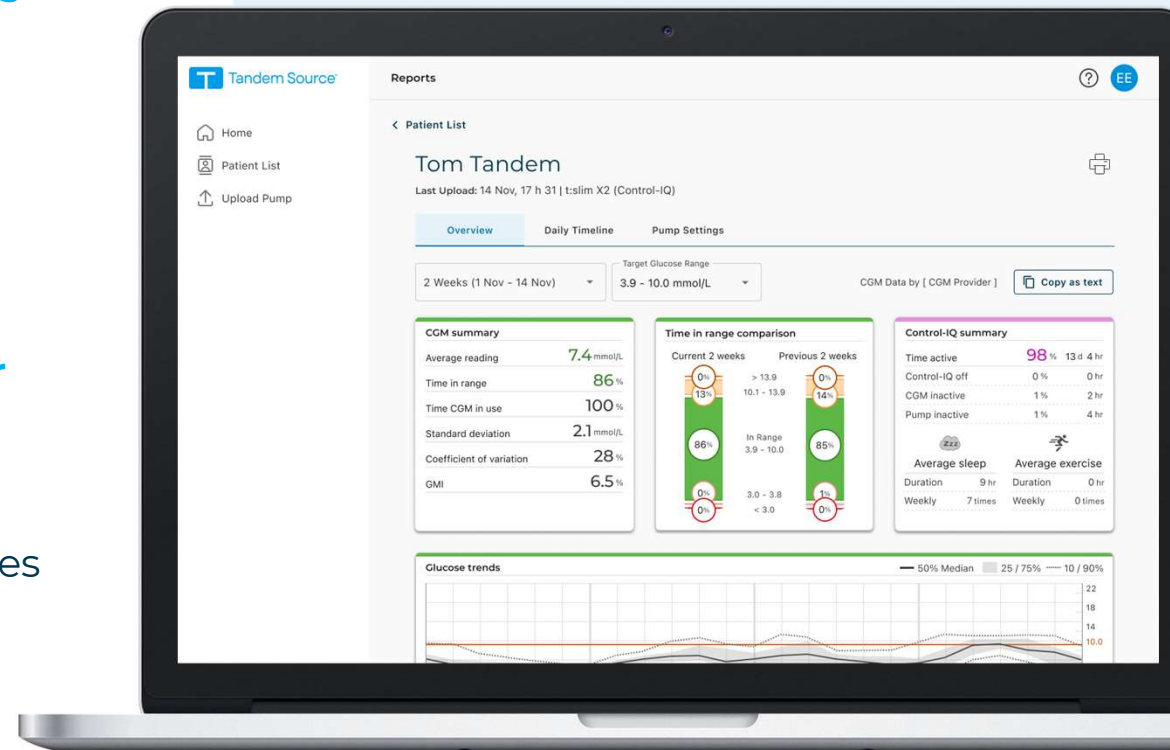
CGM summary

Average reading	7.1 mmol/L
Time in range	89%
Time CGM in use	100%
Standard deviation	2.1 mmol/L
Coefficient of variation	29%
GMI	6.4%


Time in range comparison

Current 27 days	Previous 27 days
0.8% > 13.9	1.0% > 13.9
8.3% 10.1 - 13.9	8.4% 10.1 - 13.9
89% In Range 3.9 - 10.0	88% In Range 3.9 - 10.0
1.5% 3.0 - 3.8	1.8% 3.0 - 3.8
0.5% < 3.0	0.8% < 3.0

Control-IQ summary

Time active	97%	26 d 3 hrs
Control-IQ off	0%	0 hrs
CGM inactive	1%	9 hrs
Pump inactive	2%	12 hrs

Average sleep	Average exercise
Duration 9 hrs	Duration 3 hrs
Weekly 7 times	Weekly 2 times

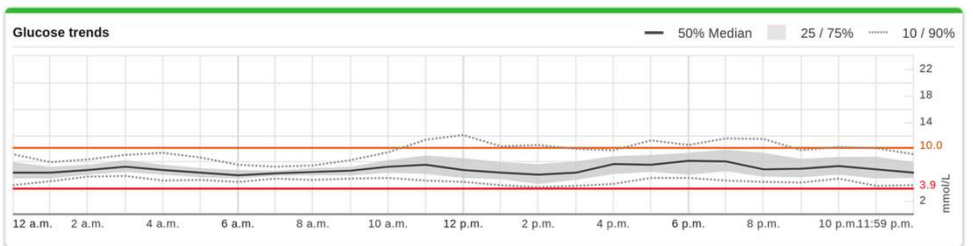Insulin summary

Average daily dose	33.92 u
Basal	49% 16.54 u
Bolus	51% 17.39 u
Average daily boluses	7 boluses
Manual	75% 5 boluses
Control-IQ	25% 2 boluses
Average daily carbs	153 g

Bolus review (daily average)

Type		
Food	88%	15.24 u
Correction	7%	1.14 u
Override	1%	0.22 u
Control-IQ	5%	0.79 u

Delivery Method

Standard	95%	16.59 u
Extended	0%	0.00 u

Load activity

Cartridge change	every	3.9 d
Tubing fill	every	3.9 d
Cannula fill	every	3.9 d

Généralités

Glucose (mmol/L)

Bolus (u)

Basal (u/hr)

Event

Sunday - Apr 21, 2024 Time in Range: 96% Avg: 6.8 mmol/L SD: 1.5 mmol/L TDI: 34.4u Carbs: 172g

Legend: BG High, BG In Range, BG Low, Auto Populated BG, Bolus (Food, Correction, Food + Correction, Extended), Basal (Profile Delivered, Profile Setting, Temporary, Control-IQ), Event (Carbs, Sleep, Exercise, Cartridge/Site Change, Lost CGM Connection, CGM Alert, Pump Alarm, Manual Stop, Shutdown).

Courbe Quotidienne

Pump Profile Settings - Daily

Time	Basal Rate (u/hrs)	Correction Factor (u:mmol/L)	Carb Ratio (u:grams)	Target BG (mmol/L)
12:00 a.m.	0.550	1:2.4	1:7.0	6.1
7:00 a.m.	0.700	1:2.4	1:5.0	6.1
12:00 p.m.	0.500	1:2.4	1:7.0	6.1

Total Daily Basal: 13.350 u Insulin Duration: 5 hrs Carbohydrates: On

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Merci

positively different™

Responsible Use of Basal-IQ and Control-IQ Technologies

Systems like the t:slim X2 insulin pump with Basal-IQ technology and the t:slim X2 insulin pump with Control-IQ technology are not substitutes for active diabetes management. For example, users still must bolus for meals. Basal-IQ technology is designed to predict and help prevent lows, but it cannot prevent all lows and Basal-IQ technology does not predict high glucose events. Control-IQ technology is designed to predict and help prevent low and high glucose events, but it cannot prevent all highs and lows, and Control-IQ requires that users accurately input information, such as meals and periods of sleep or exercise. The Basal-IQ and Control-IQ features rely on continuous CGM readings and will not be able to predict glucose levels and adjust insulin dosing if your CGM is not working properly or is unable to communicate with your pump. Be sure to always use your pump, cartridges, CGM, and infusion sets as instructed and check them regularly to make sure they are working properly. Always pay attention to your symptoms, actively monitor your glucose levels, and treat according to your healthcare provider's recommendations.

Important Safety Information:

RX ONLY. The t:slim X2 insulin pump, Basal-IQ technology, and Control-IQ technology are intended for single patient use. The t:slim X2 pump, Basal-IQ technology, and Control-IQ technology are indicated for use with NovoLog or Humalog U-100 insulin.

t:slim X2 insulin pump: The t:slim X2 insulin pump with interoperable technology is an alternate controller enabled (ACE) pump that is intended for the subcutaneous delivery of insulin, at set and variable rates, for the management of diabetes mellitus in people requiring insulin. The pump is able to reliably and securely communicate with compatible, digitally connected devices, including automated insulin dosing software, to receive, execute, and confirm commands from these devices. The pump is indicated for use in individuals six years of age and greater. **Basal-IQ technology:** Basal-IQ technology is intended for use with a compatible integrated continuous glucose monitor (iCGM, sold separately) and ACE pump to automatically suspend delivery of insulin based on iCGM readings and predicted glucose values. The bolus calculator is indicated for the management of diabetes by people with diabetes by calculating an insulin dose or carbohydrate intake based on user entered data. Basal-IQ technology is intended for the management of diabetes mellitus in persons six years of age and greater. **Control-IQ technology:** Control-IQ technology is intended for use with a compatible iCGM (sold separately) and ACE pump to automatically increase, decrease, and suspend delivery of basal insulin based on iCGM readings and predicted glucose values. It can also deliver correction boluses when the glucose value is predicted to exceed a predefined threshold. Control-IQ technology is intended for the management of Type 1 diabetes mellitus in persons six years of age and greater.

Warning: Control-IQ technology should not be used by anyone under the age of six years old. It should also not be used in patients who require less than 10 units of insulin per day or who weigh less than 55 pounds

Control-IQ technology and the t:slim X2 pump with Basal-IQ technology are not indicated for use in pregnant women, people on dialysis, or critically ill patients. Do not use Control-IQ technology if using hydroxyurea. Users of the t:slim X2 pump, the t:slim X2 pump with Basal-IQ technology, and Control-IQ technology must: be able and willing to use the insulin pump, CGM, and all other system components in accordance with their respective instructions for use; test blood glucose levels as recommended by their healthcare provider; demonstrate adequate carb-counting skills; maintain sufficient diabetes self-care skills; see healthcare provider(s) regularly; and have adequate vision and/or hearing to recognize all functions of the pump, including alerts, alarms, and reminders. The t:slim X2 pump, transmitter, and sensor must be removed before MRI, CT, or diathermy treatment. Visit tandemdiabetes.com/safetyinfo for additional important safety information.

t:connect mobile app: The t:connect mobile app is intended to be a secondary display for compatible Tandem Diabetes Care insulin pumps and is capable of wirelessly uploading pump data it receives into the t:connectweb application. The t:connect mobile app is not intended to control an insulin pump and is not intended to be a replacement for the information displayed on your insulin pump. Dosing decisions should not be made based on the secondary display device. The user should follow instructions on the continuous glucose monitoring system and insulin pump. This secondary display device is not intended to replace self-monitoring practices as advised by a physician.

© 2022 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, Basal-IQ, Control-IQ, t:connect, t:simulator, and t:slim X2 are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. Dexcom and Dexcom G6 are registered trademarks of Dexcom, Inc. in the United States and/or other countries. Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC. All third-party marks are the property of their respective owners. ML-1002426_J